

Etch 'N' Seal[®]

CE 0459

Enamel and Dentin Etchant

Reorder #030402550

Etch 'N' Seal[®] is a special formulation of phosphoric acid and aluminum oxalate. This single product allows you to etch enamel and protect dentin from possible over-etching. Oxalates have also been shown to inhibit demineralization and secondary caries. **Etch 'N' Seal** etches enamel and dentin simultaneously to maximize bond strength. For complete sealing of dentin tubules and optimal bond strength to dentin and enamel, we recommend **Tenure[®] Multi-Purpose Bonding System, Tenure[®] Uni-Bond[®] with Gloss-N-Seal[®]** or **Tenure Quik[®] with Fluoride**.

DIRECTIONS

1. Clean all tooth surfaces thoroughly and remove all caries.
2. Apply **Etch 'N' Seal** to the enamel and dentin for 10–15 seconds.
3. Rinse for 10 seconds and gently blot dry or air dry.
4. Sealing tubules: We recommend **Tenure Multi-Purpose Bonding System**. (Kit No. 031146000)

STORAGE

Do not expose to temperatures exceeding 75° F (24° C).
Do not freeze.



LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry,(800) 445-0345
customer service, technical
Domestic fax ordering and inquiry(800) 349-2600
International inquiry(805) 347-7990
International fax ordering and inquiry(805) 349-2607
Websitewww.denmat.com

Manufactured by Den-Mat Corporation, Santa Maria, CA 93455
U.S.A.

754211532W

DenMat[®]
ISO 9001 CERTIFIED

1-800-445-0345